


TONGUE STARS

HELPS PREVENT TONGUE THRUSTING

Improving on the popular Tongue Tamers

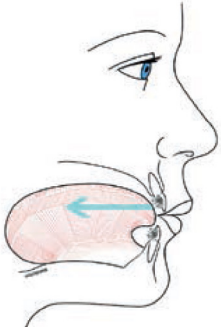

Tongue Stars have nine reminder protrusions with rounded edges to prevent anterior-lateral tongue positioning.

How Tongue Stars work

The tongue retracts from the forward positioning using a negative conditioning response, similar to a reflex action when you touch a hot object.


Tongue Stars are reminders and do not cut the tongue, or the operator's gloves or skin.

FREE
Bonding
Tweezer
with your
first order
(TOC-EO450)



Nine rounded reminder protrusions **prevents tongue thrusting**

Lingual placing - **completely invisible in the mouth**

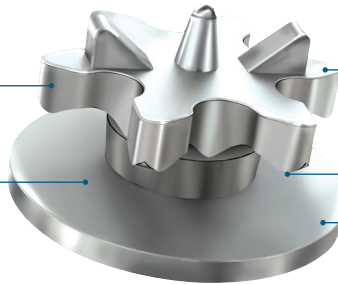

Six undercuts **for crossbite elastics in lateral open bites**

Four piece construction **provides flexibility to cope with occlusal forces**

80-gauge mesh bonding pad **for increased shear bond strength**



Placement

- Palatal of upper anteriors 3 to 3 and lingual of lower anteriors 3 to 3, bonded just below the centre of the crown to permit open bite closure.
- Bonded to the premolars and molars for lateral open bites.



Generous undercuts also allow the use of intra-oral elastics in crossbite correction. Read more about the use of Tongue Stars in rapid open bite closure by Dr John Voudouris on our website www.tocdental.com.

CODE (Pk 12)

PRICE

JT-034-10

£56.18

To order call TOC on **0117 975 5533**

E: info@tocdental.com / W: www.tocdental.com

TOC © October 2023

TOC
discover the difference